

Wash out the electrodes and WaterTabs

Disconnect in the morning after your study is complete, and before returning Seer Home to your clinic:

- ! **Important:** This process can take around half an hour, so allow plenty of time to complete this before you return Seer Home.

- 1 Take off Seer Sense – pull the connector away from the top of your head and disconnect the ECG cable from the chest stickers. Peel off the chest stickers and leave Seer Sense outside the bathroom.



- 2 Remove the WaterTabs – soak your hair under warm, running water. When the WaterTabs are sticky and gel-like, massage them out. Continue until the gel is washed out of your hair.



- ! **Important:** Avoid using hair products (e.g. conditioner) when removing the WaterTabs.

- 3 Throw away the electrodes, chest stickers, undissolved gel, and EEG headpiece.



Repeat steps 2-3 if some gel still remains.